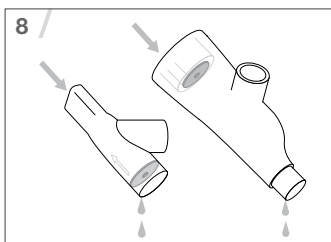
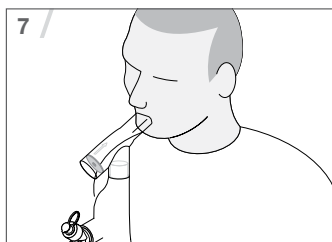
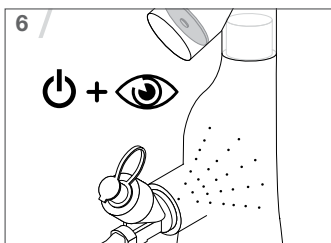
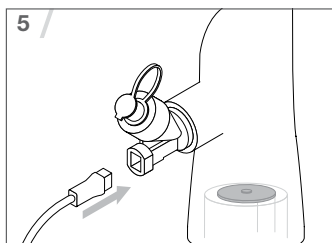
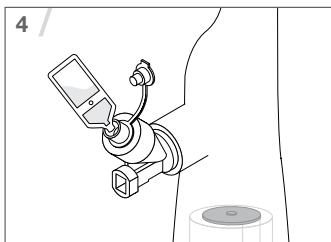
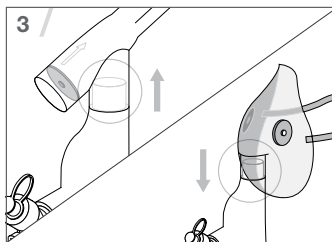
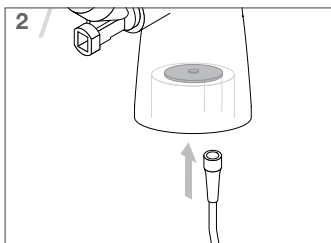
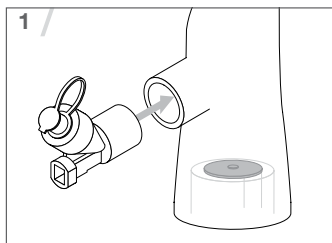
Aerogen®

Aerogen® Ultra

Instruction Manual

The Aerogen® Ultra is an accessory specific to the Aerogen Solo nebuliser. It facilitates intermittent and continuous nebulisation, with optional supply of supplemental oxygen to paediatric and adult patients via mouthpiece. The device can alternatively be used with a face mask, as supplied.

It is a single patient use device which is qualified for 20 intermittent use treatments (at a rate of four 3 ml doses per day over 5 days) or 3 hours of continuous use.

For continuous use, the Aerogen Ultra can be used in conjunction with the Aerogen Solo Continuous Nebulization Tube Set. See Aerogen Solo System Instruction Manual.

Optimal aerosol delivery is achieved with valved mouthpiece or valved aerosol face mask with low/no oxygen flow.

Aerogen Ultra Kits

AG-AS7500-EU: Aerogen Ultra Starter Kit (10 pack): Aerogen Solo, Aerogen Ultra, oxygen tubing

AG-AS7000-EU: Aerogen Ultra (10 pack): Aerogen Ultra, oxygen tubing

AG-AS7501-EU: Aerogen Ultra Starter Kit - Mask (10 pack): Aerogen Solo, Aerogen Ultra, oxygen tubing, I-Guard™ Aerosol Mask (Adult)

AG-AS7502-EU: Aerogen Ultra Starter Kit - Mask (10 pack): Aerogen Solo, Aerogen Ultra, oxygen tubing, I-Guard™ Aerosol Mask (Paediatric)

AG-AS7001-EU: Aerogen Ultra - Mask (10 pack): Aerogen Ultra, oxygen tubing, I-Guard™ Aerosol Mask (Adult)

AG-AS7002-EU: Aerogen Ultra - Mask (10 pack): Aerogen Ultra, oxygen tubing, I-Guard™ Aerosol Mask (Paediatric)

Instructions for Use - Refer to Figures 1 - 8

Inspect for device integrity and correct valve placement prior to use.

1. Insert Aerogen Solo nebuliser firmly into Aerogen Ultra in orientation shown.
2. If supplemental oxygen is required, firmly attach oxygen tubing to Aerogen Ultra.
Note: Oxygen flow rate should be set between 1-6 LPM.
3. If a face mask is required, remove mouthpiece and attach face mask to Aerogen Ultra.
Note: When using an open face mask, a minimum oxygen flow of 1 LPM is required.
4. Add prescribed medication to nebuliser as instructed in Aerogen Solo System Instruction Manual.
5. Connect cable to Aerogen Solo.
Note: Refer to Aerogen Solo System Instruction Manual for intermittent and continuous nebulisation set-up and operation instructions.
6. Power on and observe Aerogen Ultra to confirm aerosol is visible.
7. Introduce Aerogen Ultra to patient and observe aerosol flow to ensure correct operation.
8. Remove excess rainout from the Aerogen Ultra periodically (hourly with continuous nebulisation). To ensure optimum performance of the Aerogen Ultra, remove any residue by rinsing through with sterile water, shake off excess and allow to air dry.

Warnings

Performance

Do not use with a closed face mask.

When using with an open face mask, always use supplemental oxygen flow of 1-6 LPM.

Performance of Aerogen Ultra may vary depending upon the type of drug and Aerogen Ultra configuration used.

Do not exceed recommended oxygen flow for system.

Visually check Aerogen Ultra post-rinsing to ensure that valves have not become dislodged.

General

To be used by trained personnel only.

Use only with recommended components.

Ensure tubing is safely orientated to prevent strangulation hazard.

Ensure oxygen connection port or tubing is not occluded.

Do not use Aerogen Ultra without a mouthpiece or face mask.

Do not autoclave any component of the kit.

Do not cover Aerogen Ultra valves during use.

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© 2017 Aerogen Ltd.
Aerogen Ltd.